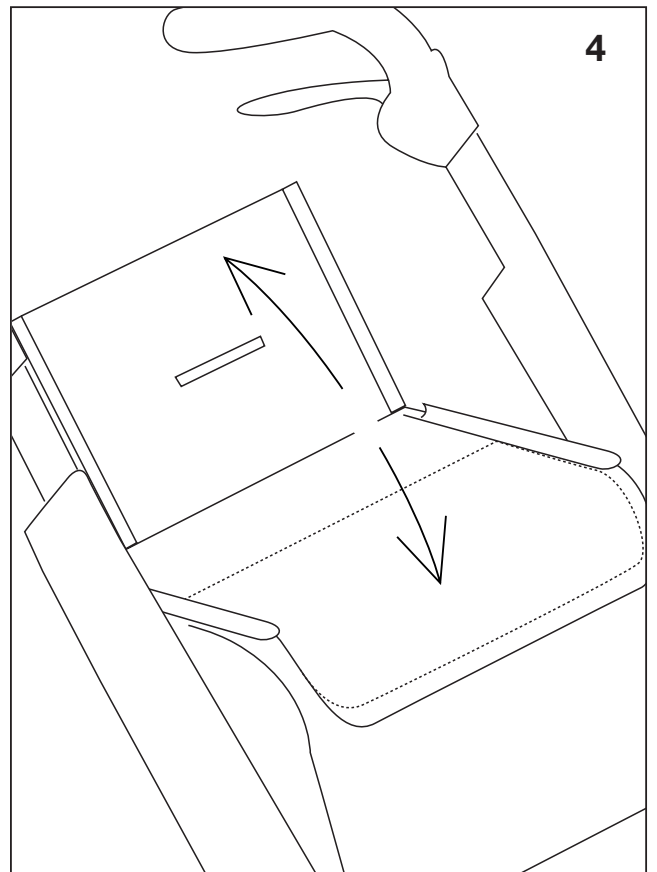
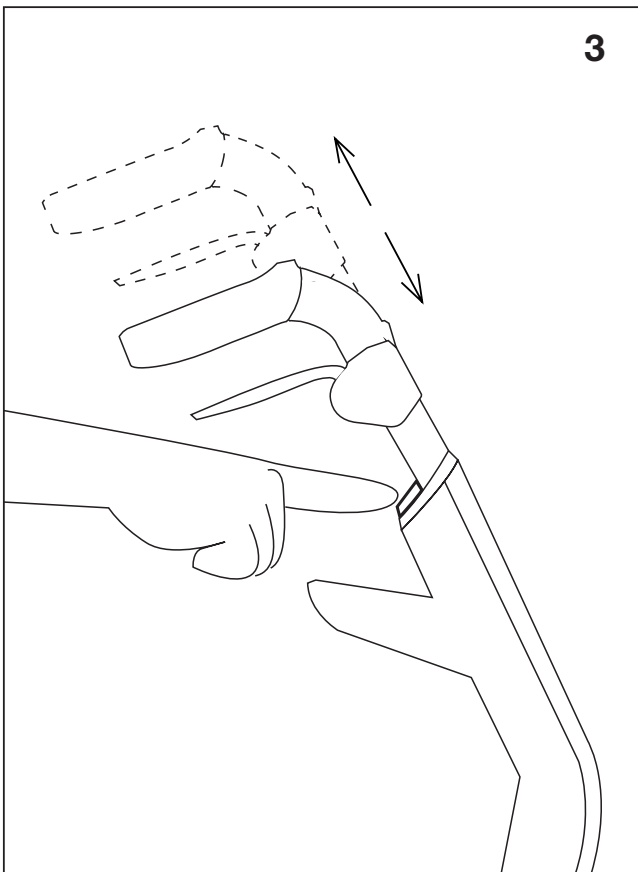
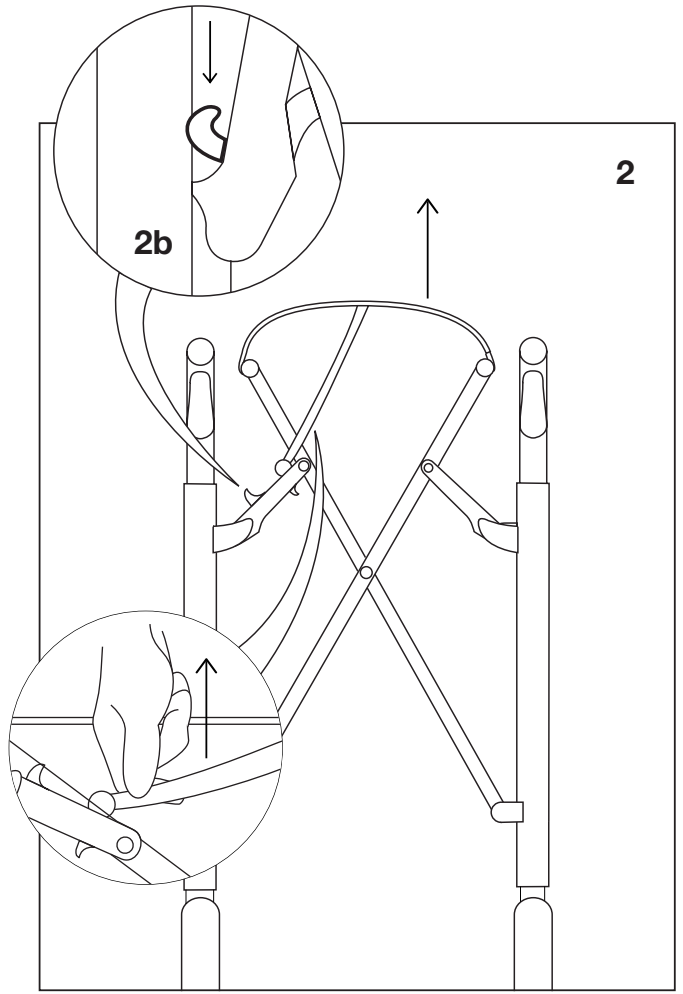
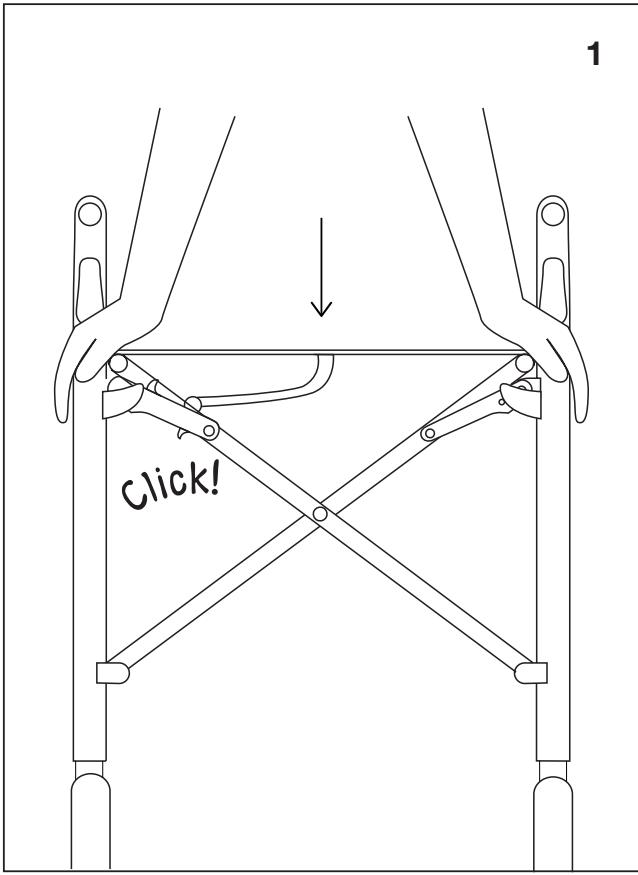
LET'S BE FREE OUTDOOR ROLLATOR

ENGLISH

Let's Be Free outdoor rollator.

We congratulate you on your new rollator, Let's Be Free, that will make life a little easier, more comfortable and safer.

The rollator is intended solely as a walking aid for person with restricted mobility. It supports persons with balance problems and limited walking ability.

To make it easier for you to become familiar with the rollator, it is important that you study these instruction carefully.

Fold the rollator for transport. Ensure that the rollator is properly secured during transport. Take extra precautions to secure the equipment during transport, if necessary.

The rollator helps you to walk and move outdoors. You can also use it to sit and rest on during longer walks. When sitting down, do not forget to activate the parking brakes - position 3 on the picture to the right.

To cross curbs and other obstacles, walk in a straight angle to the curb for the rollator not to become unsteady as it crosses the edge.

The seat is only for sitting on, not for transport for example children or objects. Don't place heavy items like boxes, furniture or similar on the seat.

Warning: Incorrect use can lead to hazardous situation - Do not use the products to transport a person.

Cleaning.

Clean the rollator with warm water and some clean soap. Do not place the rollator in places where it is hot or humid.

After use, product can be disinfected with ethanol.

Do not use the rollator in the temperatures below -35°C or above $+40^{\circ}$. Use the product in the humidity within 0-85%.

Guarantee.

We offer one year warranty. The warranty does not cover wear and tear parts like wires and wheels.

Fold out the rollator.

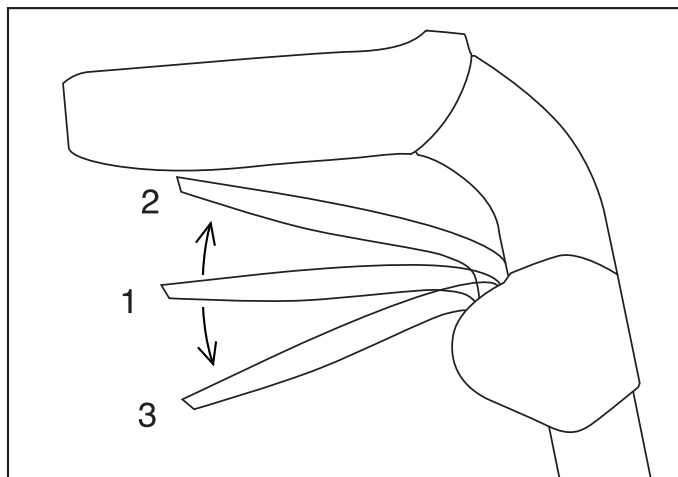
(See pictures on page 2) place the rollator upright. Push the locking hook downside (Picture 2b). Fold out by pulling the sides apart from each other. Make sure the rollator is completely un-folded and that the locking hook under the seat hooks.

Folding.

Lift the seat upwards so that the hook under the seat loosens and the rollator collapses. Press the sides against each other if necessary.

Adjustment of handle height.

Press the height adjustment latch on both sides. Set the desired handle height and check that the latch locks properly.



Open and close the bag.

Fold up the seat backwards to open the bag so that it can be loaded. Lower the seat when the bag is loaded. (See picture 4)

Brake functions.

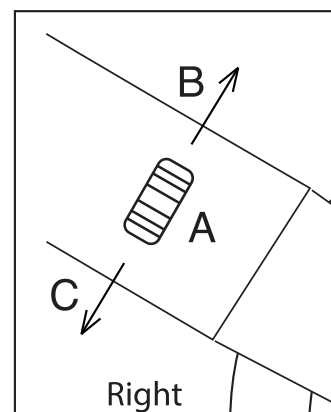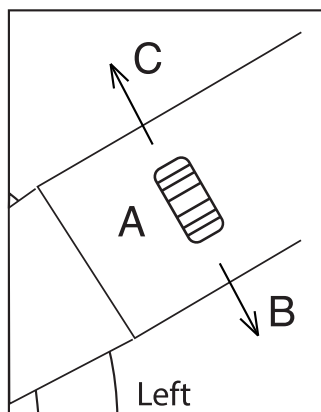1. In position 1, the rollator rolls freely.
2. To slow down the rollator, pull the handle up to position 2.
3. To park, press the handle down to position 3, where the brake locks.

Adjusting the brake.

The brakes are adjusted with the adjusting screw on the lower part of the inside of the frame. Adjust the wire tension by turning the adjusting screw (B-C). If the brake is too loose, turn in the direction of (B). If the brake is too hard, turn in the direction of (C).

Technical Data

Tested according to ISO 11199-2:2021
For a maximum user weight of 130 kg.

Width	585 mm.
Length	645 mm.
Weight	5.2 kg.
Handle height can be adjusted from	785 to 930 mm.
Seat height	580 mm.
Diameter of front wheel	200 mm.
Diameter of rear wheels	200 mm.
Max. load in the bag	12 kg.

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Maintenance

Check the function of the rollator before each use.

Check the following points:

The brakes are fully functional.

The brake cables are not twisted or torn.

The wheels are fully functional.

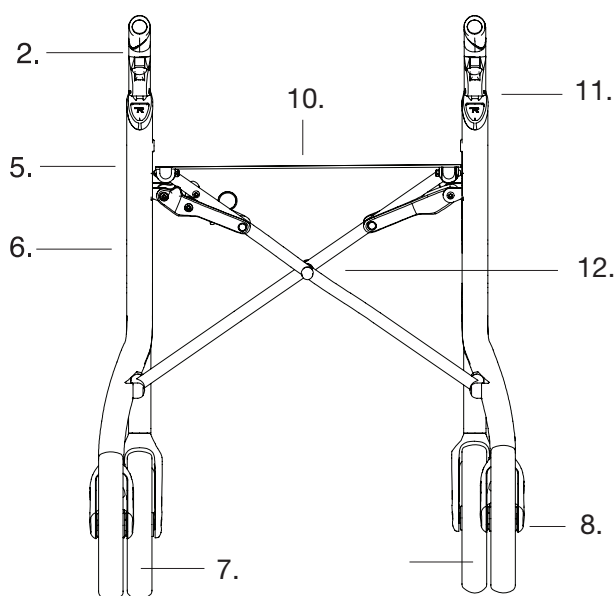
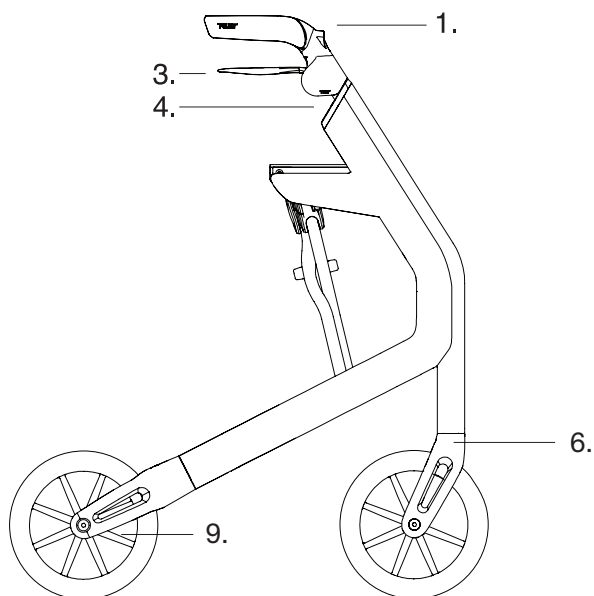
All wheels turn properly and the front wheels do not swivel.

The tyres show no signs of heavy wear.

All screws are tightly fastened.

The entire rollator is free of damage, cracks and heavy soiling.

The catch below the rollator seat is free of damage and dirt. The catch functions perfectly.

Återvinning / Genbrug / Recycling / Recycling / Recycling / Riciclaggio / Recyclage

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This product should be recycled in accordance with national guidelines.

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